

Serial-USB Adapter installation

1) Installation

- **Windows 8:** Check that your computer is connected to the internet. Plug the adapter in and wait for Windows to install the driver automatically. Follow step 2).
WARNING the « CHANNEL+ » adapter (black, U232-P9) is not compatible with Windows 8
- **Other Windows (XP 32 bits, Vista 32bits, Seven 32Bits or 64bits) :**

Do not plug the adapter in before installing the driver

WARNING: If you already have a Serial/USB adapter of any brand connected to your computer, please contact us.

Do not use the provided installation CD

While following those steps, contact us if something unexpected happens (except usual warning questions from Windows).

Use the driver provided by Framiral (download via FRAMIRAL website – instructions available in the « Find_the_Installation_Files_on_our_website » file):

Unzip the "USB-Serie.zip" file

- Start the executable file from the following folder (depending of the adapter brand):
 - KONIG (transparent with black connector) : Folder "Driver_P"
 - TrendNet (blue or white) : Folder "Driver_P"
 - CHANNEL+ (black, U232-P9): Folder "Driver_T"
 - other: Folder "Driver_T"

Follow instructions from setup.

Connect the adapter into the USB port (with nothing connected on the other end – serial).

Windows will display several message boxes on the bottom left, and possibly a warning window (click continue).

The last message should confirm the correct installation of the device – ready to use.

2) Utilization:

You can now connect the serial connector of the adapter to the serial connector of the Framiral device (Multitest/Rotational Chair/VS...).

You need to find the Serial Port number:

- Open the device manager:
 - Maintain the « Windows » key from the keyboard and press « R », then release
 - Write `devmgmt.msc` and press « enter »
- Click on the "+" on the left of "Ports (COM et LPT)"
- You will find "USB Serial Port (COMx)" or « Prolific USB-to-Serial Com port (COMx) » where "x" represents the number of the Serial COM port. Write this number down.
- Check that every COM number is unique in the list.

MULTITEST 6, FAUTEUIL 1.x, VS 2.x :

- Install the software (see appropriate installation manual)
- Start the software
- When you start for the first time, go to the Option menu / Parameters and select the serial port number written down earlier.