

Multitest 5 – End of support

Last update : September 2014

Multitest 5 software is not supported by Framiral anymore.

It has been replaced by Multitest 6 software since 2011. For more information, please contact our technical support. You may need a platform physical update, if compatible, to update your software.

Please note:

- Multitest 5 only works under Windows XP
- Multitest 5 only installs on Disk C:
- Multitest 5 only connect via COM1 or COM2
- If Multitest 5 bug when opening, try to delete the following files:
 - C:\Program Files\multitest\recntf.txt
 - C:\Program Files\multitest\equilib.cfg

You need to set the COM port at next opening.

Installation and use when not on Windows XP 32 bits:

- Installation under Vista and more recent fails. You can continue installation, but the software will be unstable.
- Feedback exercises will then have a severe bug (black font on black background).
- Some data, and patient files can be stored in the 'Virtual Store' (usually C:\Users*USERNAME*\AppData\Local\VirtualStore)
- Installation under 64bits system can cause more problems.